




FRV-RC2-PE1-197WH Specification Sheet Face Shield Assembly Kit


- Kit includes everything required for one complete face shield assembly
- Can be integrated onto Hard Hats
- Shield made from clear, Anti-fog APET material
- Shield dimensions are 197mm x 347mm
- Utilizes an adjustable strap for secure fit and universal sizing
- Foam padding on inner headband
- Intended for Multi-Use
- Meets ANSI/ISEA Z.87.1-2010 face shield standard for splash and droplet protection
- Warning: Use only as physical barrier from liquids. Face shield is not impact resistant and should not be used for eye protection.
- Combine with other forms of personal protective equipment (mask, goggles, gowns)
- Made in the USA

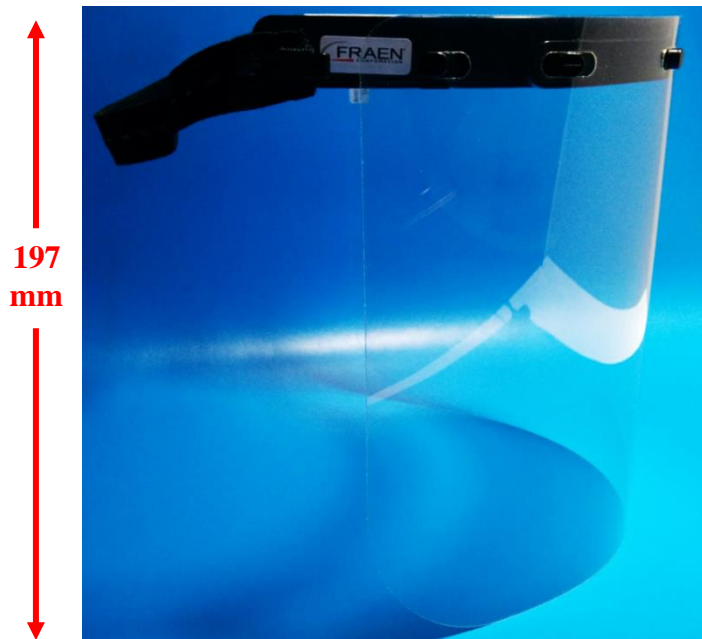




Materials List

ITEM #	DESCRIPTION	MATERIAL
1	HEAD BAND	BLACK ABS
2	FOAM LINER	CLOSED CELL, CROSSLINKED POLYETHYLENE FOAM
3	ELASTIC STRAP	Polyester Woven Elastic
4	SHIELD	.76mm Thick APET with Anti-fog coating

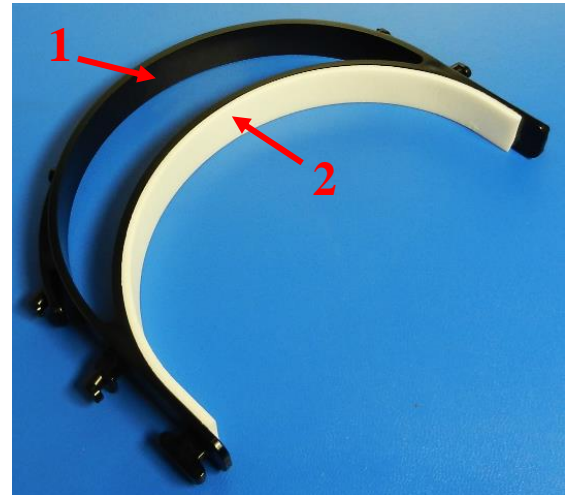

May attach directly to hard hat (not included)

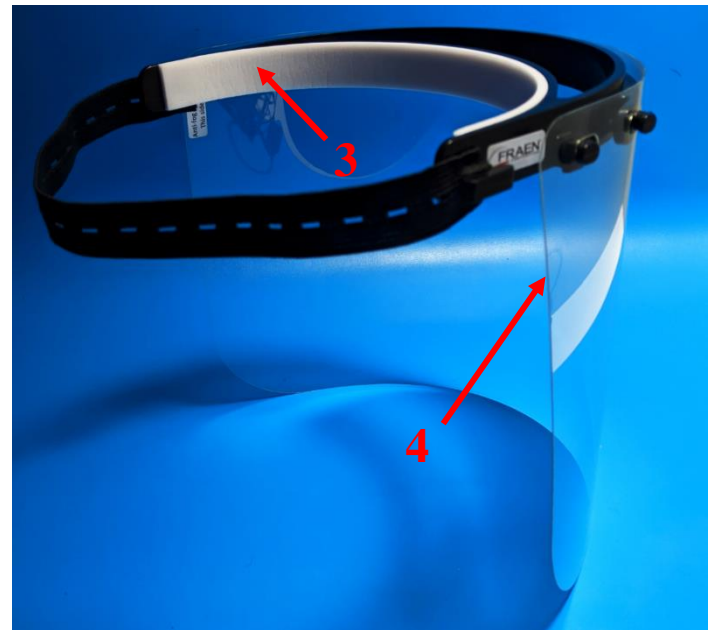



Recommendations for Use

Face shield is to be used in accordance with CDC recommendations to cover the front and sides of the face and provide barrier protection. Face shield is intended as a multi-use item. Components may be cleaned by the following means.

Components to be completely soaked, or wet, on all surfaces

Component	Method	Conditions
Shield	Soap/detergent + water	Air dry or pat with soft, microfiber cloth. *Rubbing shield during washing or drying will destroy anti-fog coating
Head Band & Liner	Bleach (sodium hypochlorite)	0.5% (1:10 bleach: water), 2 mins
	Gamma radiation	10 Mrads maximum
	Soap/detergent + water	Air dry
Elastic Strap, Woven	Launder	Wash / Standard Dry Cycle
Elastic Strap, Polyurethane	Isopropyl alcohol	75-99%, 5 minutes
	Bleach (sodium hypochlorite)	0.5% (1:10 bleach: water), 2 mins
	Gamma radiation	10 Mrads maximum

The US NIH and EPA recommend numerous disinfectant products for use with respect to Covid-19, including products containing isopropyl alcohol, bleach (sodium hypochlorite), hydrogen peroxide and soap and water. For more information please see the following: https://3dprint.nih.gov/sites/default/files/models/supplemental_documentation/IFU_Face_Shield%20Rev.1.1_03_30.20_0.pdf

<https://www.epa.gov/pesticide-registration/list-n-disinfectants-use-against-sars-cov-2>

Notes: Kit components are assembled in a clean, not sterile environment. The product has been authorized by the FDA under a EUA as personal protective equipment and has not been FDA cleared or approved.



Contact and Ordering Information

Part number: **FRV-RC2-PE1-240WH**

E-mail inquiries: CMDsales@fraen.com

FRAEN WARRANTS THAT THESE FACE SHIELDS ARE MADE IN A WORKMAN LIKE MANNER AND FREE FROM MANUFACTURING DEFECTS. FRAEN DISCLAIMS ALL OTHER EXPRESS AND IMPLIED WARRANTIES, INCLUDING WITHOUT LIMITATION ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. IN THE EVENT A FACE SHIELD IS DEFECTIVE, BUYER'S SOLE REMEDY IS PRODUCT REPLACEMENT.

*Published by Fraen Corporation.
Last revised July 2, 2020*